



Vantage Review

How to Review an Image Set

1. Open Optos V² Vantage Pro Review
2. Select “User Name”
3. Select “Show patient history”

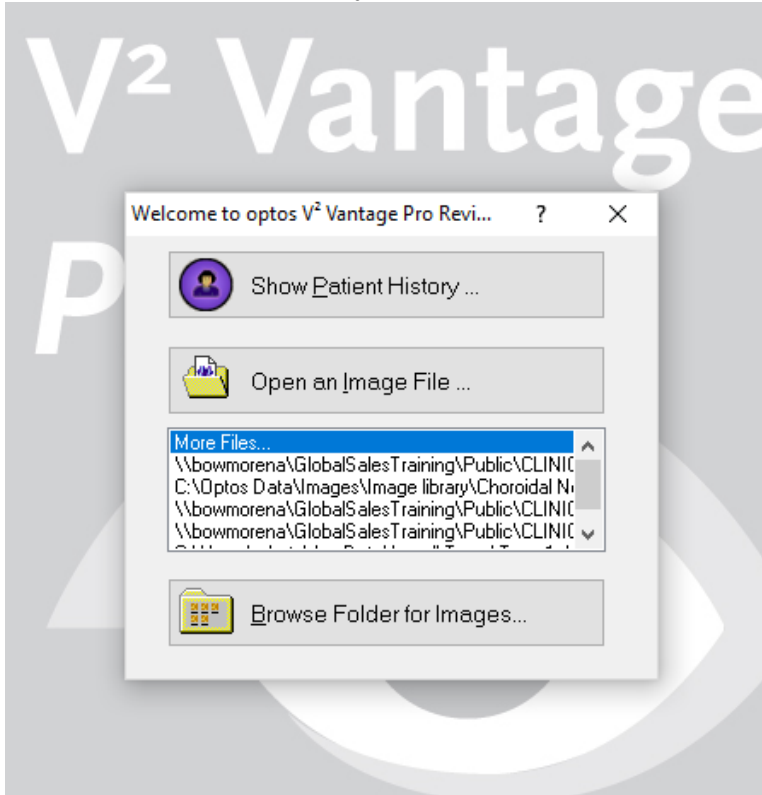


4. Close out of the select patient window by selecting the **X** at the top right
5. On the tool bar select **Review** and choose **Image Set**

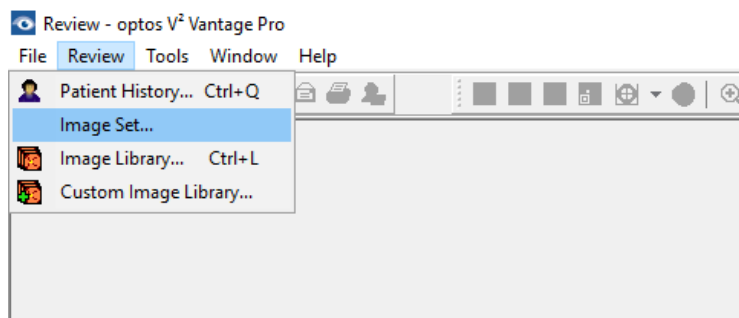


6. At next menu select the drop-down box that says “Last Year” to view the drop-down menu. Choose from Any Date, Date Range, Today, Yesterday, Last Week or Last Month.



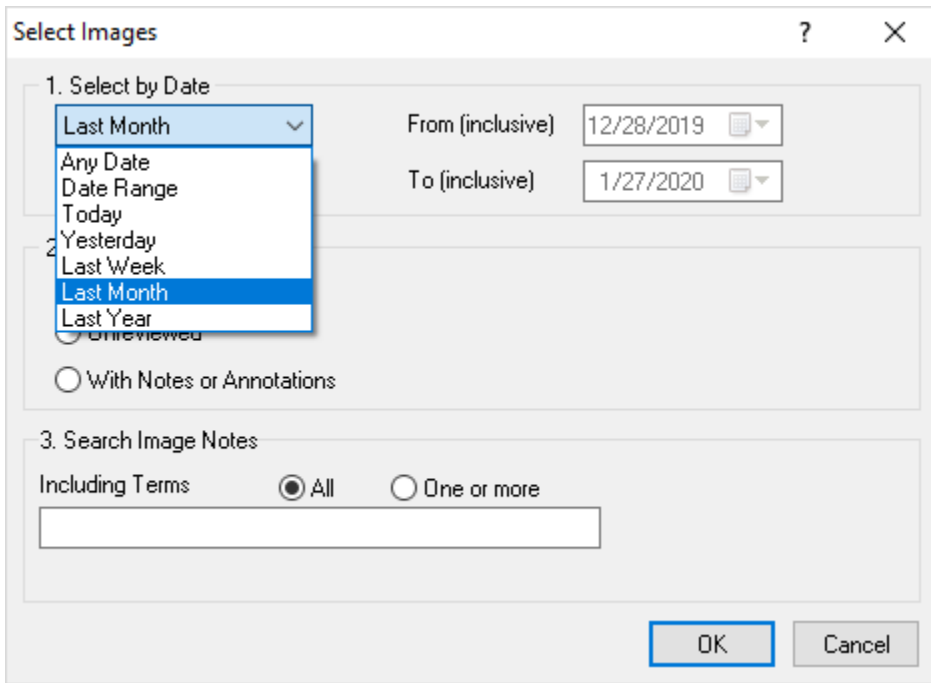

7. Select **OK**
8. You should see a screen like this with thumbnail images from dates selected.

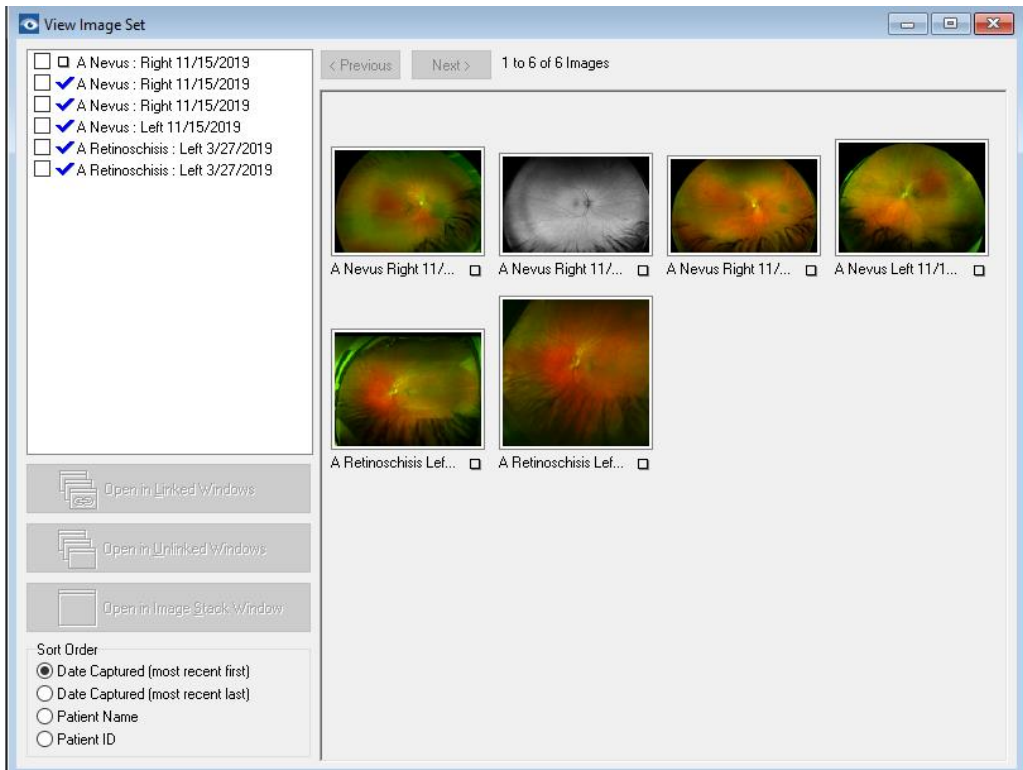

9. When completed click the **X** button on the top right.

