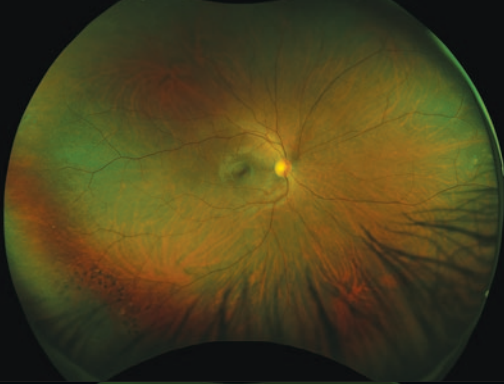
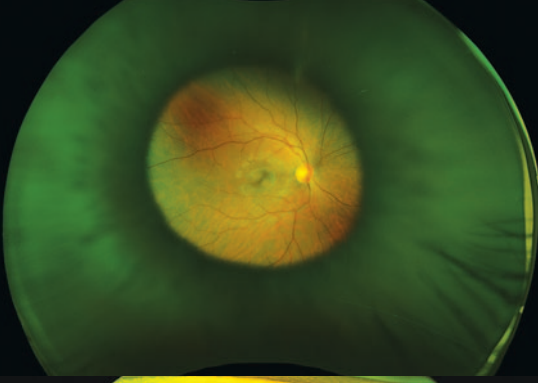
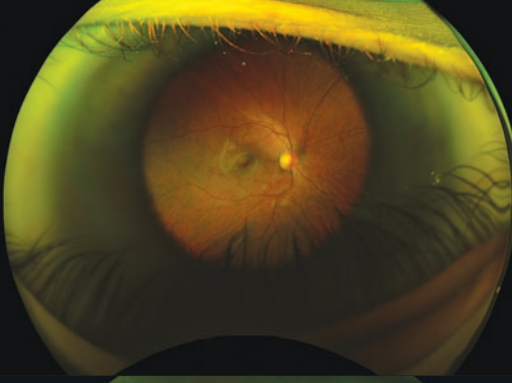
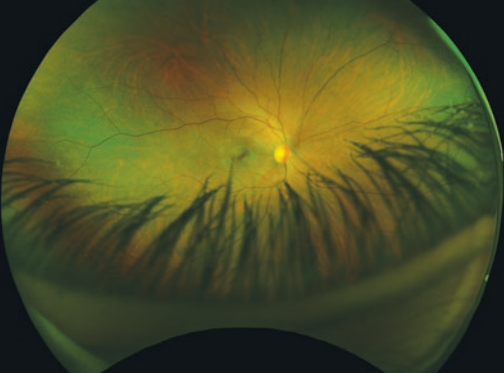
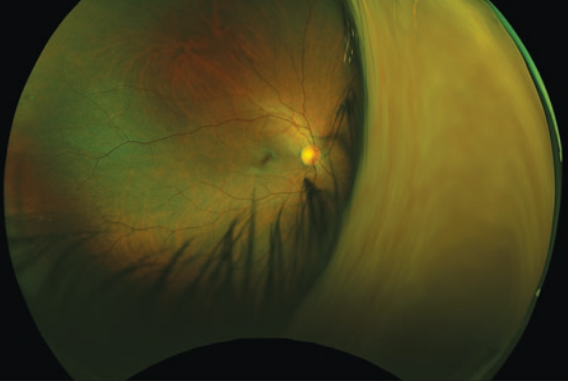
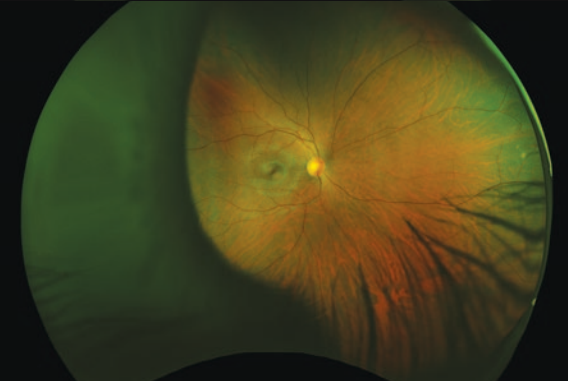
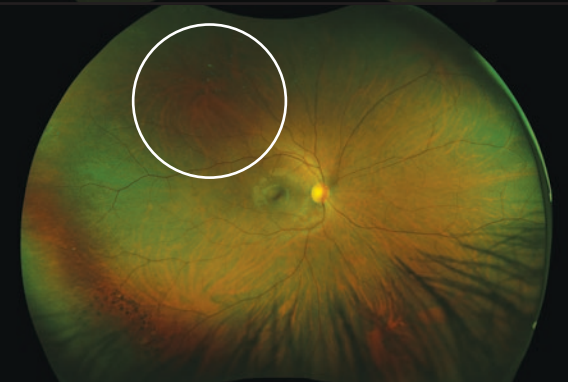
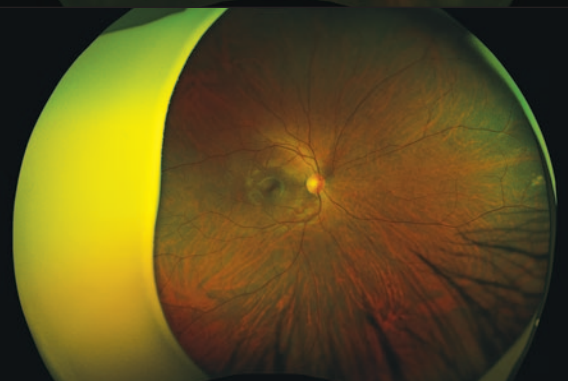
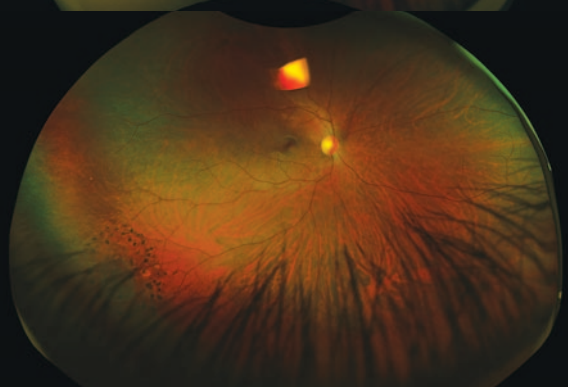


| | | |
|----------------------------|--|--|
| <p>Correct Positioning</p> |  | <p>This is a good image.</p> |
| <p>Too Far Out</p> |  | <p>Patient is too far out, the target is blue.</p> <p>Position patient until target turns green.</p> |
| <p>Too Close</p> |  | <p>Patient is too close, the target is red.</p> <p>Position patient until target turns green.</p> |
| <p>Too Much Eyelid</p> |  | <p>Have the patient open both eyes as wide as possible. The height of the table may also need to be raised up.</p> |

| | | |
|----------|--|--|
| Blink |  | Have the patient keep both eyes open as wide as possible. |
| Movement |  | Patient moved during imaging or the pupil is not centered. Position the patient so the target is in the center of the pupil. |
| Dust |  | Dust has accumulated on the main mirror. Clean mirror per protocol. |
| Facepad |  | Patient nose is pressed against facepad. Turn the patient's head slightly to the side. |
| Reflex |  | Patient is too close or not centered. Reimage. |