

OptosAdvance™ Image Preview



Purpose

This document provides guidance on how to use the Image Preview in Patient Directory for OptosAdvance software version v24 and beyond. Image Preview allows you to see a thumbnail of images contained in a study from the Patient Directory screen.

NOTE: Full operating instructions are provided in the Help section of OptosAdvance. Once logged into the software, select the **Help** icon from the toolbar to search or browse guidance topics. To confirm your version of the software, hover over the OptosAdvance logo.




Contents

Purpose.....	1
Image Preview Settings	1
Turning Image Preview Off	2
Using Image Preview	2

Image Preview Settings

Image Preview is a feature that can be turned off or on. To change these settings, login into the software with administrative permissions.

- 1 | Once logged in, select the gear Settings icon in the upper right corner.
- 2 | In the Search field, type **Patient Search Study Table** and select that menu option.

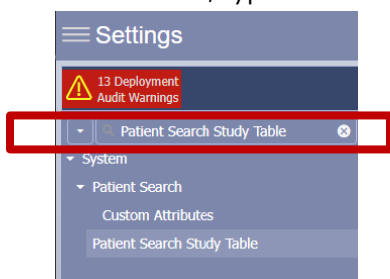

- 3 | Look at the bottom of the screen to view your options. If 'Show series preview on single click' is selected, the Image Preview is on. Leave **Load Study Mode** set to double click on study row.



OptosAdvance™ Image Preview



Turning Image Preview Off

- 1 | Deselect the Show series preview on single click.



- 2 | **Select Load Study Mode** to single click on study row.
- 3 | Select **Save** to save your settings.

Using Image Preview

Use Patient Search to find the patient and study you want to review.

Single click the study to open the Image Preview. The thumbnails of image modalities contained in that study will display in the bottom of the screen.



- 1 | Single click any thumbnail to preview the image.
- 2 | You can Zoom, Pan, and Scroll through the preview.
- 3 | Select the X to close out the preview window and select another thumbnail.
- 4 | To open the study for full image review functionality, double click the study in the Patient Directory.

